


VariVac^{™*} Wound Drainage System

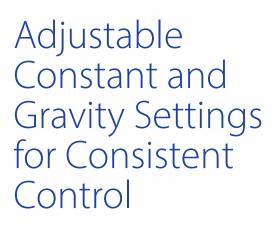



AUSTRALIAN

VariVac^{™*} gives you the control that you need in a wound drainage system

EASY TO ADJUST VACUUM

Designed for all surgical cases, simply dial the amount of vacuum desired from approximately 0 kPa to 15 kPa.

CONSTANT VACUUM

Will maintain the vacuum you require to its capacity.

GRAVITY / PASSIVE SETTING

This unique feature allows you to not only minimise excessive drainage but also return to normal drainage when required by simply adjusting the dial.

CRYSTAL CLEAR

Reservoir container allows for easy identification of exudate flow.

PRE-VACUUMED AND RECHARGEABLE

For ease of use and convenience.

INFECTION CONTROL

Many features include oneway valve, quick connectors, strong polycarb construction for maximum safety and protection.

TOTALLY CLOSED VACUUM SYSTEM

Safe containment of exudate eliminates aerosol spray.

VACUUM INDICATOR


Large, easily visible for identification of vacuum present in reservoir.


AUSTRALIAN DESIGNED

Designed by an Australian company. Australian Patent Reg. No. 36448/95



The innovation Gravity back to normal drainage by just turning the dial.

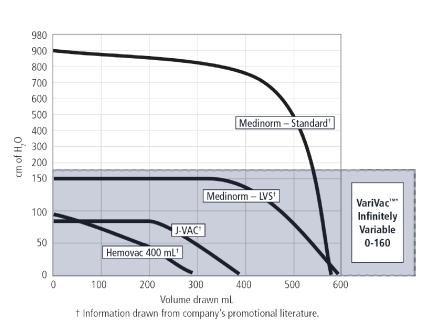

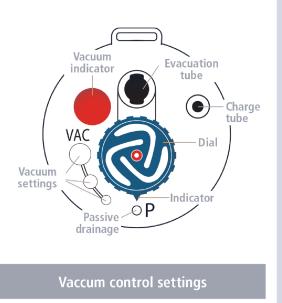

Vacuum adjustment

To regulate the vacuum level, simply turn the vacuum dial.

Range is adjustable between the approximate settings below:

Passive (Gravity)	0 kPa
Low	5 kPa (37 mmHg) / 51 cmH ₂ O
Medium	10 kPa (75 mmHg) / 102 cmH ₂ O
High	15 kPa (113 mmHg) / 153 cm \tilde{H}_2 O

VariVac^{™*}

Australian made wound drainage system

Product Code	Description	Qty
VC 500-02	10 Fr (3.2 mm, 1/8") kit complete	20
VC 500-03	15 Fr (4.8 mm, 3/16") kit complete	20
VC 500-04	19 Fr (6.4 mm, 1/4") kit complete	20
VC 500-11	500 cc Reservoir kit Y connector	20
VC 500-12	500 cc VariVac ^{™*} reservoir only	20
VC 501-22	Varivaca manual pump with filter	Each
VC500-61	Varivac ^{™*} Combo with VariBlake 10Fr with Trocar and connecting tubing with connector	20
VC500-62	Varivac ^{™*} Combo with Variblake 14Fr with Trocar and connecting tubing with connector	20
VC500-63ST	Varivac ^{™*} Combo with Variblake 19Fr with Trocar and connecting tubing with connector	20
VC500-64	Varivac ^{™*} Combo with Variblake 8mm Flat Drain with trocar	20
VC500-65	Varivac ^{™*} Combo with Variblake 11mm Flat Drain with trocar	20

Ascend Surgical & Medical

Ascend Surgical and Medical Unit 9/3 Leo Alley Road Noosaville, Queensland 4566 Contact: 1300 784 893 Contact the team at Ascend Surgical and Medical for support with the VariVac™ range of products.

www.ascendsurgical.com.au